





# TestSure

## At Home Drug Test Kit

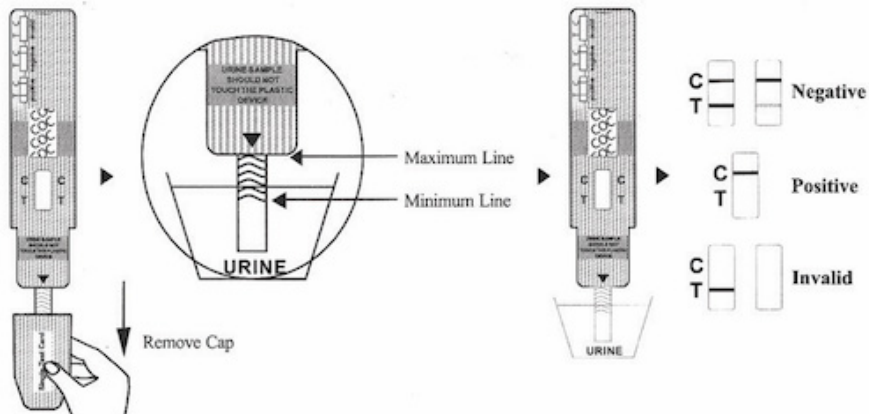

   @testsuredotcom

MORE INFO  
[www.testsure.com](http://www.testsure.com)

## INSTRUCTIONS

1. Remove the TestSure Test Kit from the sealed pouch and use it as soon as possible. Remove the cap from the end of the test card. With arrows pointing toward the urine specimen, immerse the strip of the test vertically in the urine specimen for at least 10-15 seconds. **DO NOT ALLOW URINE TO TOUCH THE PLASTIC.**
2. Replace cap and place the TestSure Test card on a non-absorbent flat surface. Start the timer. Read the adulteration strips between 3-5 minutes and compare the colors on the adulteration pads to the enclosed color chart. If the specimen indicates adulteration, refer to your Drug Free Policy for guidelines on adulterated specimens. We recommend not to interpret the drug test results and either retest the urine or collect another specimen.
3. The drug strip results should be read at 5 minutes.

## INTERPRETATION OF RESULTS

**NEGATIVE\*:** Two lines appear. A colored line appears in the Control region (C) and a colored line appears in the Test region (T). This negative result means that the concentrations in the urine sample are below the designated cut-off levels for a particular drug tested.

**\*NOTE:** The shade of the colored line(s) in the Test region (T) may vary. The results should be considered negative whenever there is even a faint line.

**POSITIVE\*:** A colored line appears in the Control region (C) and NO line appears in the Test region (T). The positive result means that the concentration in the urine sample is greater than the designated cut-off levels for a particular drug tested.

**\*NOTE:** Insufficient specimen volume or incorrect procedural techniques are the most likely reasons for Control line failure. Read the directions again and repeat the test with a new test card. If the result is still invalid, contact TestSure to determine the issue.