

TestSure 

www.testsure.com

KETONE TEST STRIPS

ANTI-VC
INTERFERENCE ABILITY

SUITABLE FOR PEOPLE WITH
DIABETES OR LOW CARB DIETERS

TEST YOUR BODY KETONE
LEVELS IN JUST SECONDS

100 STRIPS

Please Read

1. Do not store in direct sunlight.
2. Do not remove desiccant.
3. Replace cap immediately and securely after each use.
4. Use bottle within 6 months after first opening.

**For In Vitro Urine Testing Only.
Always consult your physician
before use.**

Directions

1. Remove strip from bottle. Immediately replace cap.
2. Do not touch test panel on strip.
3. Dip test panel into urine sample and remove it immediately.
4. Wait for 40 seconds and then match test panel results to Ketone color chart on bottle.
5. Ignore color changes that occur after 40 seconds.

FDA CE

