



www.testsure.com

Urinalysis Test Strips

10 Parameters

Anti-VC interference ability

Diagnostic test kit for Urine Chemistry. For *in vitro* use only.

IMPORTANT:

- Keep away from light and moisture.
- Promptly replace cap after taking out strips.
- Do not remove desiccants.
- Do not touch test areas of reagent strips.
- Use it within the expiration date.
- Read insert carefully before use.



100 Strips TestSure

TESTING AND READING TIME



Leukocytes 120s	Neg.		Trace 15	Small 70	Moderate 125	Large 500	cells/ μ l	
Nitrite 60s	Neg.			Positive Any degree of uniform pink color				
Urobilinogen 60s	3.2	Normal	16	32	64	128	μ mol/l	
Protein 60s	Neg.		Trace \pm	0.3 +	1.0 ++	3.0 +++	g/l	
pH 60s	5.0	6.0	6.5	7.0	7.5	8.0	8.5	
Blood 60s	Neg.	Non hemolyzed 10 Trace		Hemolyzed 10 Trace	25 Small	80 Moderate	200 Large	cells/ μ l
Specific Gravity 45s	1.000	1.005	1.010	1.015	1.020	1.025	1.030	
Ketone 40s	Neg.		Trace 0.5	Small 1.5	Moderate 4.0	8.0	Large 16	mmol/l
Bilirubin 30s	Neg.			Small 17	Moderate 50	Large 100	μ mol/l	
Glucose 30s	Neg.		5 Trace	15 +	30 ++	60 +++	110 ++++	mmol/l

DIRECTIONS FOR USE

This product is very easy to use. Simply fill a cup with urine and immerse the urine test strip into the sample. Remove from the urine sample, and lay it down on a smooth flat surface. In minutes, you'll be able to read results.

Use the chart above to read the test results. If any levels seem too high, or if you're unsure, it's best to consult with a doctor to find the cause of the high levels.

Shelf life: 24 months

Storage: Room Temperature