

TestSure 
www.testsure.com

Reagent Strips for Urinalysis
Urinary Tract Infection Test Strips

IMPORTANT:
Keep away from light and moisture.
Promptly replace cap after taking out strip.
Do not remove desiccants.
Do not touch test areas of reagent strips.
Use it within the expiration date.
Read insert carefully before use


LOT



100 Strips



Testing And Reading Time

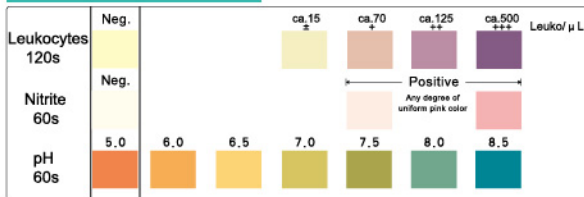

Test Procedure:

1. Remove one strip from the bottle and replace the cap immediately.
2. Immerse the reagent area of the strip in the urine specimen and take it out quickly.
3. Wipe off excess urine against the rim of the specimen container.
4. Read the test results carefully within 60 seconds in a good light and with the test area held near the appropriate color chart on the bottle label. Changes in color that appear along the edges of the test pads after more than 2 minutes have passed are of no diagnostic significance. Results for leukocytes test portion can be read within 120 seconds if read instrumentally. Carefully follow the directions given in the appropriate instrument operating manual.

Reading Results:

1. Nitrite Positive:
It is positive when the reaction pad becomes pink in color within 1 minute. If urine tests positive for Nitrites then a urinary infection is present.
2. Leukocytes Positive:
It is positive when the reaction pad becomes purple in color within 2 minutes. If urine test positive for Leukocytes then the urinary system has suppurative inflammation.
3. Higher than normal levels (above 7) of pH in the urine specimen could be sign of urinary tract infection.